

## Electric-powered esthetic salon bed

This product is intended for use in esthetic treatments, including facial care, body care, and hair removal.




Model No.

**AY-C030B**  
(Upper Part)

**AY-C03ENB**  
(Lower Part)

**EVRI-1N**

## Operating Instructions

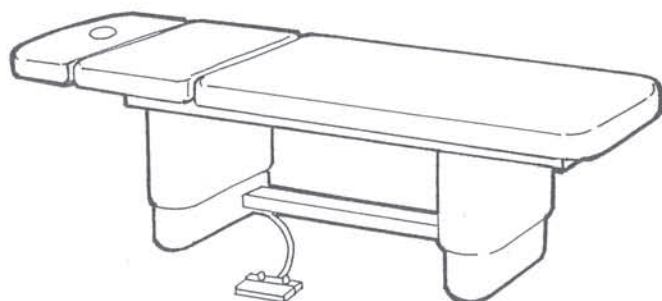

## Table of Contents

Description of Warning Symbols	1
Safety Precautions	1
Warning Label Location	3
Part Names	4
How to Use	4
Method of Operation	5
Maintenance Method	7
Standard Period of Use	8
Specifications	9

Thank you for purchasing this Takara Belmont product.

**Please read the Operating Instructions carefully before use and use the product appropriately.**

Thereafter, please keep the Operating Instructions in a safe place and read them again whenever necessary. Failing to read the Operating Instructions before use may lead to accidents.

This product is a commercial device intended for use in aesthetic and beauty salons for their treatments and services.

This product must be operated or handled by the qualified as an esthetician/hairdresser or by their staff under the supervision of an esthetician/hairdresser.

Salon customers should not be allowed to operate or handle the product unless he or she is so instructed.

September 2025 (Version 1)

1E0B5QA0(Inquiry No.)



# Description of Warning Symbols

## The precautions shown on this page






are intended for the safe use of the product and to prevent harm and damage to you and other people. All of the precautions shown are important details related to safety. Please be sure to follow these precautions.

Detailed descriptions of symbols are provided below. Read and understand these symbols thoroughly before reading these instructions.





The precautions that must be complied with are categorised and explained using the following symbols.

 <b>WARNING</b>	Explains precautions that may result in death or serious injury.
 <b>CAUTION</b>	Explains precautions that may result in mild or moderate injury or property damage.

The precautions that must be complied with are categorised and explained using the following symbols.

<b>Prohibition</b> (Actions that are not allowed)	<b>Mandatory</b> (Actions that must be carried out)
  General prohibitions      Do not disassemble.	   “Mandatory” actions that must be carried out      Disconnect the power plug.      Connect the ground conductor.

# Safety Precautions

 <b>WARNING</b>	
 <b>Make sure to install a grounding conductor.</b> <p>There is a risk of electric shock in the event of a mechanical failure or electrical leakage. Consult a professional to perform proper connection to the electrical ground.</p>	 <b>Comply with the following when handling a power cord.</b> <ul style="list-style-type: none"> <li>• To unplug, hold and pull the power plug, not the power cord.</li> <li>• Do not use if the power plug or cord is damaged or if the power plug is not tightly inserted into the outlet. An electric shock or fire may result.</li> </ul>
<b>Never touch the inside of the main unit.</b>	
 <b>Never touch the inside of the product. There is a possibility of electric shock.</b> <p>For maintenance and service of the product (incl. power plugs and cords ), be sure to contact your local authorized Takara Belmont dealer.</p>	

## ⚠ WARNING



**Unplug the power plug if the product will not be in use for a long time.**

Make sure to unplug the power plug from the outlet at the end of each work day or during holidays when the product will not be in use for a long time. Insulation may deteriorate, causing fire due to electrical leakage.



**Avoid disassembling or repairing the product by yourself under any circumstances.**

Avoid disassembling, repairing, and/or modifying the product by anyone other than a repair technician under any circumstances. There is a risk of fire or abnormal activation resulting in injury.



**Comply with the following when handling a power plug.**

- Use a separate AC 230 V outlet.
- Insert the power plug fully into the outlet.



**Children shall not play with the appliance.**

Children being supervised not to play with the appliance.



**This appliance is not to be used by the following individuals.**

he appliance is not to be used by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction.

## ⚠ CAUTION



**Install the product on a stable surface.**

Install the bed on a level, solid floor. The bed may tip over. Slight backslashing due to unevenness of the floor surface can be reduced with level adjusters.



**Unplug by holding the power plug.**

Unplug the power plug from the outlet by holding the plug. Unplugging by pulling the power cord may cause disconnection.



**Have a client lie so that her/his hip comes at the junction of the backrest and the bed.**



Unless a client lies so that her/his hip comes at the junction of the backrest and the bed, excessive force may be applied to her/him when the backrest and the headrest are raised or lowered, resulting in injury.



**Do not sit or apply weight on the edge of the bed or on the headrest.**



Do not sit or apply weight on the edge of the bed or on the headrest. The bed may tip over, resulting in injury.

## ! CAUTION



**Move the backrest after the headrest fixation knob is fully inserted in place.**



Place the headrest so that the headrest is level with the backrest, and fully insert the headrest fixation knob before moving the headrest. Forcibly moving the headrest may cause a malfunction.



**Do not raise the backrest while sitting on it.**



Do not raise the backrest while sitting on it. Excessive weight may be applied to the backrest and may cause a malfunction.



**Check to see if there is anyone or anything under the bed before starting raising or lowering operations.**



Check to see if there is anyone or anything under the bed before raising or lowering the seat. Serious injury may occur by pinching, and malfunction may result.




**Do not raise the backrest when an object/objects or a hand/hands are between the backrest and the lower part of the bed.**



Make sure that an object/objects or a hand/hands are not between the backrest and the lower part of the bed before raising the backrest. If a hand/hands get caught, injury may result, causing the bed to malfunction. In the case of children, serious injury may occur.



**Perform a monthly inspection on threaded**

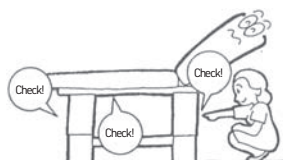


Long-term use may cause threads, nuts, bolts, etc., to become loose. Perform a monthly inspection on threaded parts to check for looseness. If any part is loose, tighten it firmly with a screwdriver. If the part is still loose after retightening, please contact your local authorised Takara Belmont dealer.



**Keep an eye on the client and any children.**


- Keep an eye on the client while using this product. If the client (especially a child) touches the control panel built into the side of the armrest or the foot lever or causes the product to activate accidentally by accidental contact, an unexpected accident may occur.
- Keep children away from this product except while you offer services.

## Warning Label Location

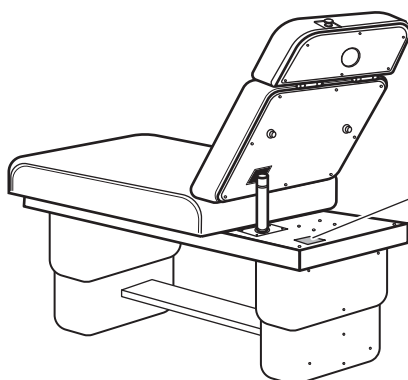



# Part Names

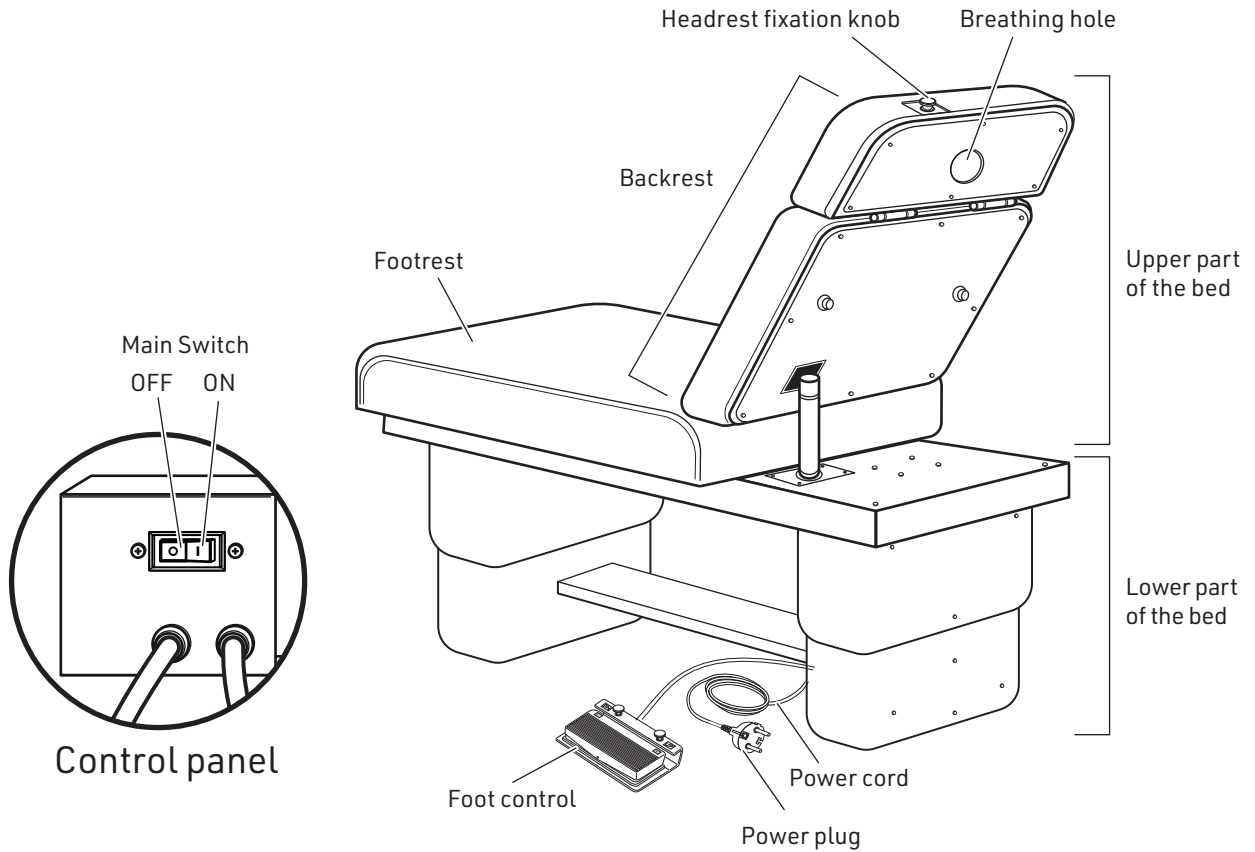

# How to Use

- (1) Insert the power plug into the outlet.
- (2) Main switch pressing the right side (marked 'I') of the main switch will turn the power on.
  - The power switch of this bed has a built-in circuit-breaker function.
- (3) Have a client sit down so that her/his hip comes at the junction of the backrest and the bed.
- (4) When it's done, the main press the left side (marked 'O') of the main switch make sure to unplug the power plug from the outlet when the product will not be in use for a long time.




## WARNING

Unplug the power plug from the outlet by holding the plug. Unplugging by pulling the power cord may cause disconnection.


# Method of Operation

## When raising the entire bed

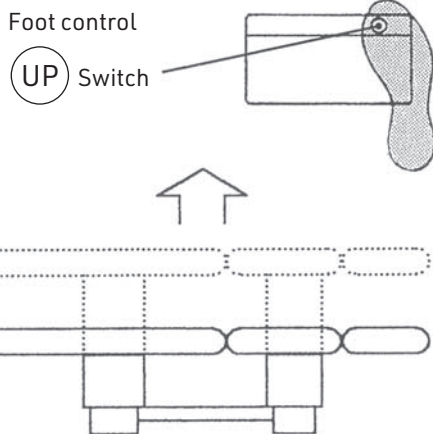

Step on the **UP** switch lightly, and the bed will rise to its upper limit.  
 ( To stop midway through, step on any of the four buttons. )

## When lowering the entire bed

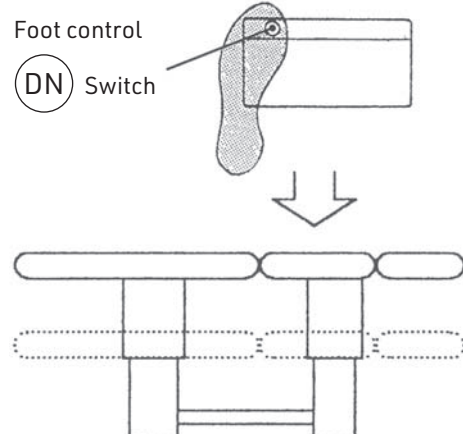


Step on the **DN** switch lightly, and the bed will lower to its lower limit.  
 ( To stop midway through, step on any of the four buttons. )

## To adjust the degree of rising/lowering

To raise slightly, depress the **UP** switch twice in a row.


To lower slightly, depress the **DN** switch twice in a row.

## To raise the backrest

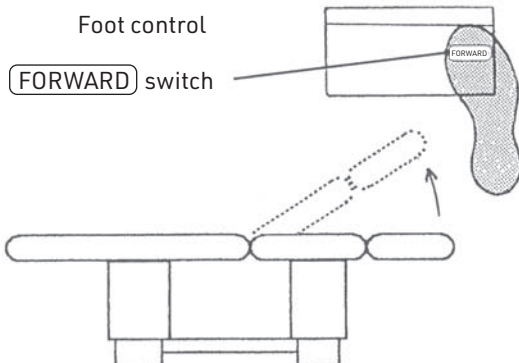

When the **FORWARD** switch is depressed, the backrest will be raised.  
 Releasing the foot will stop the back section at that position.

## To lower the backrest

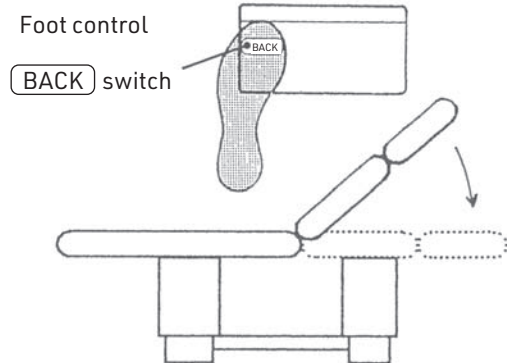


When the **BACK** switch is depressed, the backrest will be lowered.  
 Releasing the foot will stop the back section at that position.

If the headrest fixation knob is not fully inserted in place or if the headrest is bent downward, the backrest does not move.

When lowering the backrest, make sure to raise the headrest and firmly insert the headrest fixation knob.

## When bending the headrest

You can bend the headrest downward by supporting the headrest with one hand and pulling the headrest fixation knob in the direction of the arrow with the other hand.

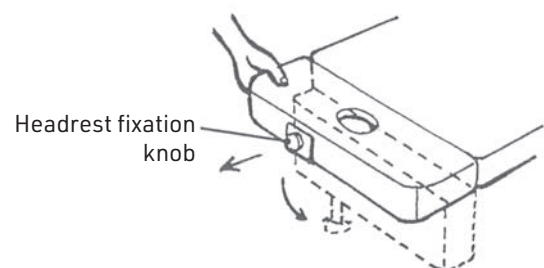

## When raising and securing the position of the headrest

Lift the headrest until it is level with the backrest, and insert the headrest fixation knob.

# Method of Operation


## For safe use of "EVRI-1N"

If the product is used with an imbalance in load between the head section and the leg section of the bed, or if the product is moved by holding the bed, a difference (gap) may occur between the heights of the head section and leg section of the bed.



### CAUTION

- At the beginning of daily use
  - When the bed does not move properly
- Raise the entire bed to the upper limit.

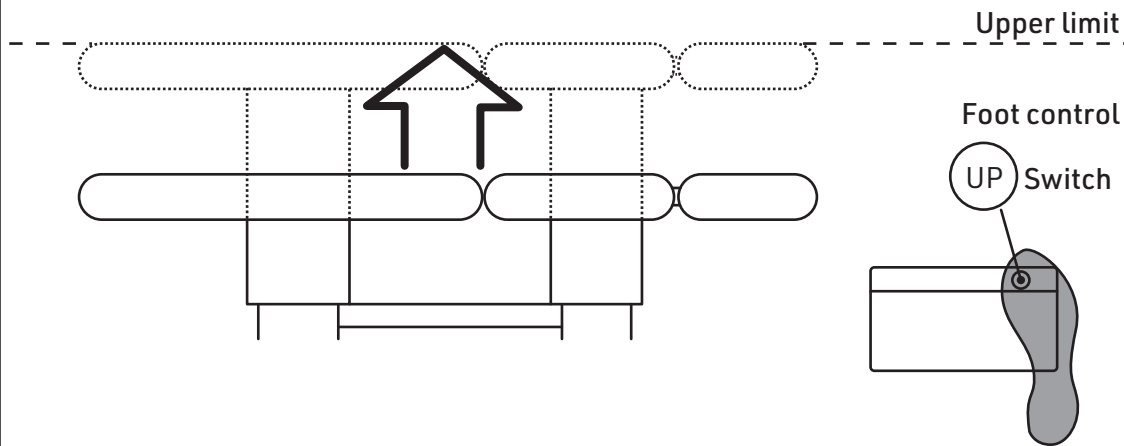

#### [Method of Operation]

- Make sure nothing is placed on the bed.
- Step lightly on the UP switch of the foot control, and raise the bed to its upper limit.

After reaching the upper limit, the operation will continue for about three seconds, so please wait until the operation sound stops. Thereafter, use the product at the desired height.

#### [Effect]

The operation eliminates the difference in height between the head section and the leg section of the bed.


#### [If this operation is not done...]

malfunctions may be caused, such as the bed not moving properly or not coming down to the lower limit.

# Maintenance Method

## Vinyl leather parts

Wipe with a damp, soft cloth.

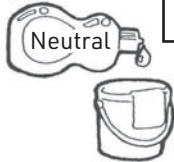

### For severe staining

Wipe with a dilute neutral detergent.

## Metal parts

Wipe with a dry, soft cloth without applying anything.




Avoid moisture and water.



 **CAUTION**

## The following materials must not be used:


Volatile chemicals, such as thinner, alcohol, gasoline, kerosene, wax containing abrasives, and sponges containing abrasives

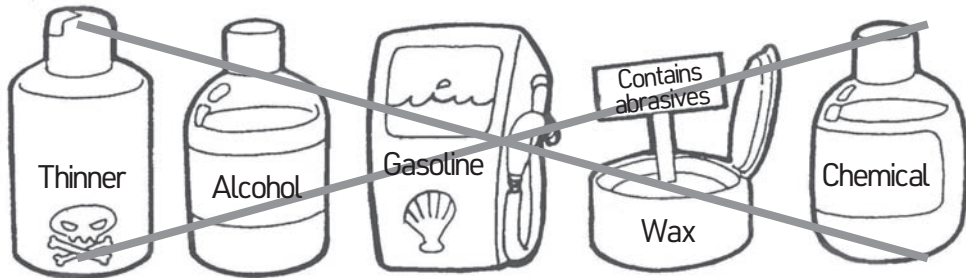

## Standard Period of Use

- The standard period of use for this product is ten years.
- Standard period of use is defined as "a guideline for the standard period of using the product without any problems in safety under standard conditions of product use with appropriate maintenance and inspection (including replacement of parts) performed." Please be advised that this is not an indication of the guarantee period, and the standard period will differ depending on the environment in which you use the product and the frequency of use. Please contact your local authorised Takara Belmont dealer for maintenance.

# Specifications

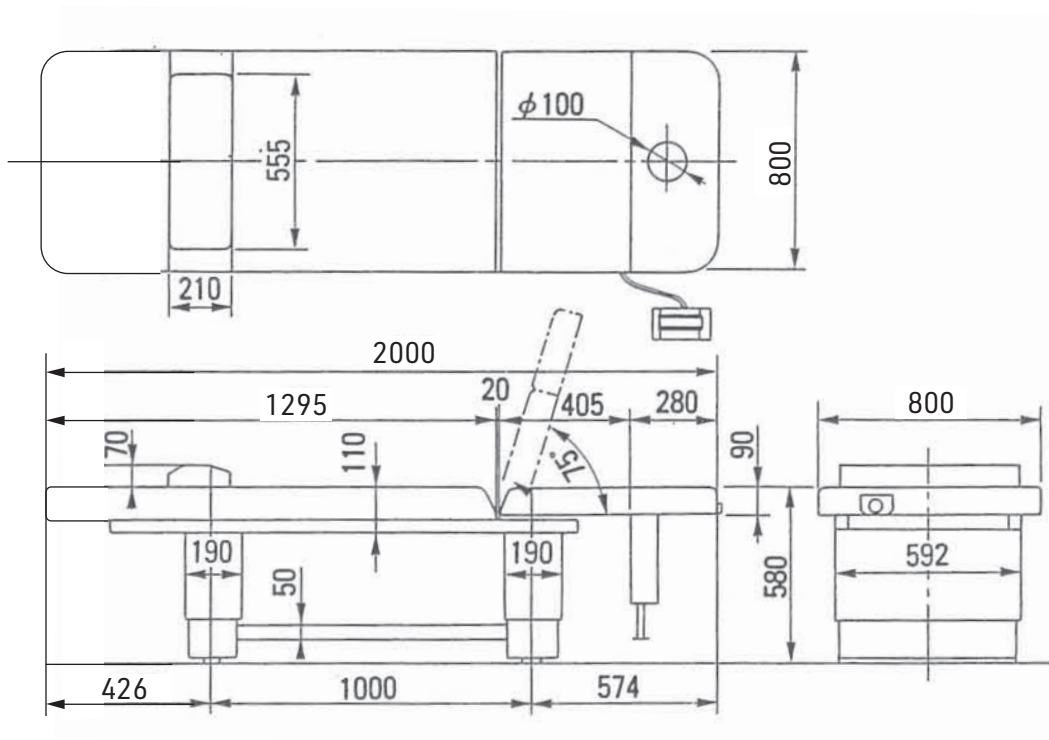

Total length ..... 2000 mm x total width 800 mm x total height 575-785 mm

Stroke ..... 210 mm (initial height 575 mm)

Weight ..... approximately 122 kg

Rated power consumption ..... 230 V (50 Hz) / 1.7A

## MEMO

# IMPORTER INFORMATION

COUNTRY		REGISTERED PLACE OF BUSINESS & ADDRESS
GBR	Importer in UK	TAKARA BELMONT (U.K.) LTD. BELMONT HOUSE, ONE ST.ANDREWS WAY, BOW LONDON E3 3PA, U.K.
DEU	Importeur in Deutschland	TAKARA COMPANY EUROPE GmbH Industriestrasse 21, 61381 Friedrichsdorf, Germany
FRA	Importateur en France	TAKARA COMPAGNIE PARIS S.A.R.L. 56 rue des Hautes Pâtures 92009 NANTERRE Cedex France
ITA	Importatore in Italia	TAKARA BELMONT ITALY S.R.L. Via Marino Mazzacurati No.4/1, 42019 Scandiano RE, Italy

## EU/UK Declaration of Conformity

We declare under sole responsibility that the device that is covered by the present declaration is in conformity with the following directives (and regulations), and, if applicable, with any other relevant EU and UK legislations that provides for the issuing of an EU/UK DECLARATION OF CONFORMITY.

Product : ESTHETIC TREATMENT TABLE

Model : EVRI-IN

Regulations this product conforms to:

EMC (electromagnetic compatibility) Directive : 2014/30/EU,

Machinery Directive : 2006/42/EC

RoHS Directive : 2011/65/EU based on category 11 of Annex I.

The product has been designed and manufactured in accordance with the European standards as the listed in the Declaration of Conformity.

This declaration relates exclusively to the machinery in the state in which it was placed on the market, and excludes components which are added and/or operations carried out subsequently by the final user.

EU Authorised representative

TAKARA COMPANY EUROPE GmbH

Industriestrasse 21, 61381

Friedrichsdorf, Germany



UK Authorised representative

TAKARA BELMONT (U.K.) LTD.

BELMONT HOUSE, ONE ST.ANDREWS WAY, BOW

LONDON E3 3PA, U.K.



TAKARA BELMONT CORPORATION

2-1-1, Higashishinsaibashi, Chuo-ku, Osaka, 542-0083, Japan